


## Face Shields: Protective Face Shield Visor



**Clear face shields protect health care providers, essential services workers and consumers.**

- Shield made from 10mil clear PET
- Elastic headband for flexible sizing
- Foam cushion for extra comfort

**Part # FS-001.**

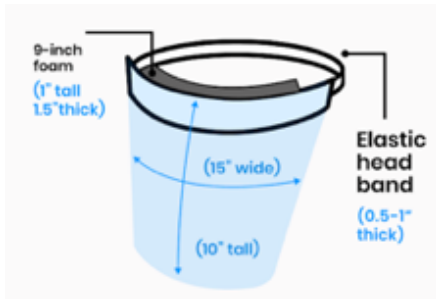

### Wearing the Face Shield

To put the face shield on, stretch the elastic around your head so that the foam sits on your forehead. This product is suitable for individual use only. Do not share it with others.



### Cleaning the Face Shield

Clean both sides of the shield with soap and warm water after each use. Sanitizer can be applied with a paper towel. Aggressive cleaning agents are not recommended. Store in a safe, clean, and dry area, such as a Ziploc bag. Dispose after 7 days of usage.



### A Front Line Essential

**Who can use these shields?**

- Health care providers
- Emergency response teams
- Animal health care providers
- Retail, grocery store, and service employees



PET shield meets FDA requirements under 21 CFR 177.1630.  
EVA filler complies with FDA regulations 21 CFR 177.1350,  
Paragraphs (a), (c), (d), and (e).

